


VERSA II SETUP FOR NON-REBREATHING (for patients 20 lbs or less)

Please note, the flush bypasses non-rebreathing mode. Therefore, regardless of the position that the switch is in, when the flush button is pressed, the oxygen will only enter the rebreathing system. To fill the non-rebreathing circuit bag, use the oxygen flowmeter.



Switch is flipped DOWN for non-rebreathing.

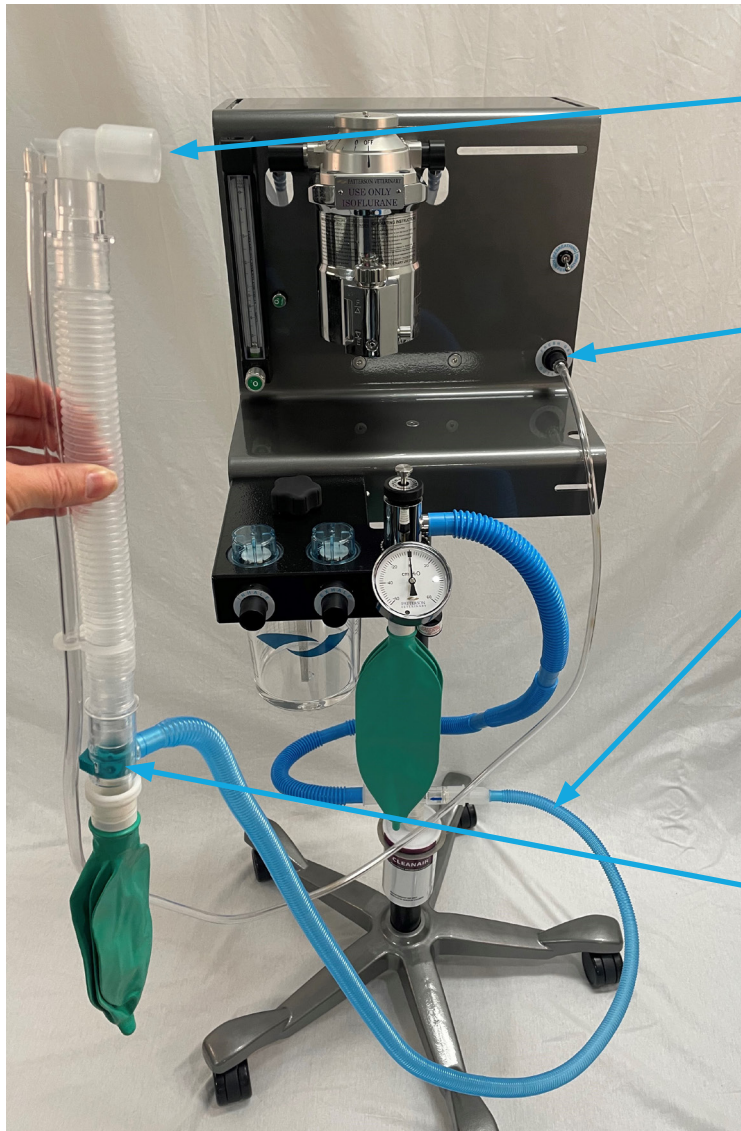

Connect to patient

Smooth clear tubing on non-rebreathing circuit is plugged into port labeled FRESH GAS OUTLET.

Blue corrugated tubing on non-rebreathing circuit attaches to scavenger (F/air filter).

Open/Close valve should always remain OPEN unless you are administering a manual breath to the patient. When finished administering a manual breath, valve should be reopened immediately.

For support, please contact Patterson Veterinary Anesthesia and Monitoring hotline at 866.825.6076.


PATTERSON
VETERINARY